LiteScope™ laryngoscope

Instructions For Use

The instruction for use is not supplied individually. Only 1 copy of the instructions for use is provided per box and should therefore be kept in an accessible location for all users.

NOTE: Distribute instructions to all product locations and users.

These instructions contain important information for the safe use of the product. Read these instructions for use in their entirety, including warnings and cautions before using this product.

WARNING: A WARNING statement provides important information about a potentially hazardous situation, which if not avoided, could result in death or serious injury.

CAUTION: A CAUTION statement provides important information about a potentially hazardous situation, which if not avoided, could result in minor or moderate injury.

Product Description

The Intersurgical LiteScope™ is a single use, one size, fully disposable laryngoscope for use in adult patients. Product code 7124000.

Indications For Use

The LiteScope is designed to provide direct visualisation of the larynx. It helps to facilitate oral tracheal intubation in adults by personnel who are suitably trained and experienced in the use of airway management techniques and devices, including intubation.

Intended use

To facilitate laryngoscopy.

Contraindications

- Surgical airway.
- Fluid or solid material in the airway that cannot be cleared with suction.
- Trismus, limited mouth opening, pharyngo-perilaryngeal abscess, trauma or mass.
- Operator inexperience with laryngoscopy.
- Use in an MR environment.

Warnings

- For use by qualified medical personnel only. Operators must familiarise themselves with the LiteScope and the instructions for use before using the device.
- No modification of this device is permitted.
- The device contains no user serviceable parts.
- The batteries within the device should not be removed prior to its disposal.
- No attempt should be made to reprocess, sterilise or re-use the device.
- Do not use force to insert the device.
- Particular care should be taken with patients who have fragile and vulnerable dental work, in accordance with recognised airway management practices and techniques.
- Ensure that a backup plan is in place in case of difficulty or emergency whilst performing the procedure.
- Do not use the device in strong magnetic fields such as MRI environments.
- The LiteScope is a single use product, do not reuse. Reuse poses a threat to patient safety in terms of cross infection, device malfunction and breakage. Cleaning can lead to device malfunction and breakage.
- Do not use after expiry date.

Size selection

LiteScope is available in one size for adult patients. In smaller adult patients, it may not be necessary to insert the blade as far as may be necessary for an average sized adult patient in order to gain an optimal view of the relevant anatomical structures. For patients where the blade of the LiteScope is not large enough to enable these structures to be viewed, an alternative device should be used.

Pre-use checks

- Inspect the packaging and ensure it is not damaged prior to opening.
- Inspect the device carefully and discard if the device looks abnormal or deformed.
- Turn the device on and off by depressing the on/off button (Picture 1) and check there is a visible light at the distal end of the blade. Discard the device if it cannot be successfully switched on and off.

Using the LiteScope

- Turn on the device by pressing the on/off button (Picture 1).
- Laryngoscopy should be conducted using the same technique as would be used with any standard laryngoscope with a Macintosh type blade.
- Confirmation of correct placement of the tracheal tube should be assessed by the use of waveform capnography and other methods in accordance with applicable guidelines.
- The LiteScope should then be switched off by depressing the on/off button (Picture 1).

Storage conditions

Store in a dry place out of direct sunlight.

Recommended storage at room temperature over the indicated product shelf-life.

Disposal

- The LiteScope should be disposed of in accordance with normal procedures for clinical waste.
- Please contact your local authority or Intersurgical for more information.

Size	156mm(H) x 29mm(W) x 169mm(L)
Weight	78g
Minimum Interdental Gap	24mm
Electrical rating:	6.0v DC 35mA

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Intersurgical Ltd, Crane House, Molly Millars Lane, Wokingham, Berkshire, RG41 2RZ, UK
T: +44 (0)118 9656 300 info@intersurgical.com www.intersurgical.com

Ireland
T: +353 (0)1 697 8540
info@intersurgical.ie

España
T: +34 91 665 73 15
info@intersurgical-es.com

Nederland
T: +31 (0)413 243 860
info@intersurgical.nl

Sverige
T: +46 08 514 30 600
info@intersurgical.se

Россиа
T: +7 495 7716809
info@intersurgical.ru

South Africa
T: +27 (0)11 444 7968
info@intersurgical.co.za

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Canada
T: +1 905-319-6500
info@intersurgical.ca

France
T: +33 (0)1 48 76 72 30
info@intersurgical.fr

Portugal
T: +351 219 108 550
info@intersurgical.pt

België/Belgique
T: +32 5 2456991
info@intersurgical.be

Danmark
T: +45 28 69 80 00
info@intersurgical.dk

Česká Republika
T: +420 272 911 814
info@intersurgical.cz

中国
T: +86 519 69817000
info@intersurgical-cn.com

Pilipinas
T: +63 2820 4124
info@intersurgical.ph

Colombia
T: +57 1 742 5161
info@intersurgical.com

Deutschland
T: +49 (0)2241 25690
info@intersurgical.de

Italia
T: +39 0535 20836
info@intersurgical.it

Luxembourg
T: +32 5 2456991
info@intersurgical.be

Lietuva
T: +370 387 66611
info@intersurgical.lt

Türkiye
T: +90 216 468 88 28
info@intersurgical.com.tr

日本
T: 81 (0)3 6863 4359
info@intersurgical.co.jp

USA
T: +1 800 828 9633
support@intersurgicalinc.com

Australia
T: +61 (0)2 8048 3300
info@intersurgical.com.au

Distributed in the USA by Intersurgical Incorporated, 6757 Kinne St., East Syracuse, NY 13057
T: (800) 828-9633 F: (315) 451-3696 support@intersurgicalinc.com www.intersurgical.com

patent: <http://us.intersurgical.com/support>
LiteScope™-3-H 2118 Issue 8 RO 07.21