Gebrauchsinformation / Instruction for use

Werolin® Wundreinigungsspray

MEDIZINPRODUKT / MEDICAL DEVICE

DE

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Werolin® Wundreinigungsspray is a spray for cleansing, moisturizing and healing of acute wounds.

Characteristics:

Werolin® Wundreinigungsspray is used for cleansing and moisturizing acute wounds and supports wound healing (prevention of infections due to low pH).

- ✓ **Cleansing of acute wounds, e.g. lacerations, abrasions and cuts, 1st and 2nd degree burns**
- ✓ **Decontamination of acute wounds**
- ✓ **Reduction of swelling**
- ✓ **Reduces bleeding**
- ✓ **Supports wound healing**
- ✓ **Prevents infections**
- ✓ **Optimizes the environment on the wound surface (pH-level)**
- ✓ **For the pain-free wound cleansing**
- ✓ **Solves wound crusts**
- ✓ **Suitable also for children**
- ✓ **Prevents scarring***

* if the wound area is covered with a bandage and kept moist with Werolin® Wundreinigungsspray until the wound is closed

Indications:

For acute and initial care for minor, open injuries (e.g. cuts, abrasions, lacerations) and 1st and 2nd degree burns.

Directions for use:

Cleanse wound with **Werolin® Wundreinigungsspray**. Apply directly out of the bottle into the wound and if necessary, repeat several times. **Werolin® Wundreinigungsspray** can be used in combination with customary wound dressings. If possible, cover the wound area with dressing and keep wet **Werolin® Wundreinigungsspray** until the wound is closed. Make sure that the dressing does not stick to the wound. We recommend further treatment with **WERO Wund- und Heilsalbe** from the 3rd day onwards. For a more cooling effect store **Werolin® Wundreinigungsspray** in the refrigerator before application. Can be used on the mouth and on mucous membranes. The overhead spray allows spraying from any position. Suitable also for children. The duration of use is based upon type and severity of the injury.

Contraindication:

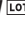
Do not use with hypersensitivity to any of the ingredients. A doctor should be consulted with burns second b or third degrees. Application for pregnant or breastfeeding women, babies and toddlers/ young children should only be carried out with mandatory indication after consultation of a doctor. Avoid contact with cartilage-tissue. Do not use in combination with anionic tensids. Do not moisten dressing materials containing silver with **Werolin® Wundreinigungsspray**.

Tissue tolerance and biocompatibility:

Werolin® Wundreinigungsspray does not contain skin-irritating ingredients. It is non-irritating and nonsensitizing.

General safety instructions:

For external use only. Do not swallow. Look after hygienic extraction. Close bottle after use to avoid contamination. Dispose the bottle immediately after contact of the bottle with the wound or in the case of otherwise contamination. Keep out of the reach of children. Consider temperature. Consider instructions for use. Do not use after expiration date. Do not use damaged packages. Protect from sunlight. Store in a dry area.

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Vertrieb durch: WERO GmbH & Co. KG · Idsteiner Str. 94 · D-65232 Taunusstein
NAWA Heilmittel GmbH · Ostendstrasse 100 · D-90482 Nürnberg