
Operation Guide for Holter Recorder and AI-ECG Analysis System

English | Español | Français | Deutsch | Italiano

Operation Guide for Holter Recorder and AI-ECG Analysis System

1. Operation Flow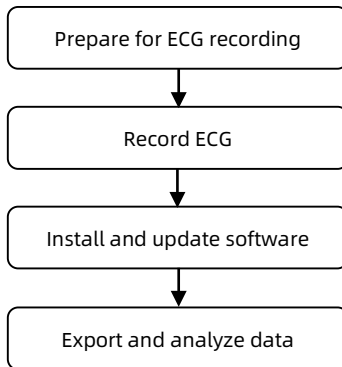

2. Operation Instructions

1) Prepare for ECG recording

1. Install new batteries and ensure its positive and negative poles are correct. After the battery is correctly installed, the recorder will start up automatically and the screen will display the startup interface.

Caution

- **DO NOT insert a battery with the positive and negative poles reversed.**
- If the device prompts "Insufficient voltage!!! Do you want to continue?", you can select "Yes" to continue or replace with new batteries. However, if you select "Yes", Holter Recorder may not be able to record the data for 24 hours.
- If the device prompts "Please replace with new batteries!", **please remove the screws from the battery cover and replace with AAA/1.5V alkaline batteries.**

2. Connect the lead wire to ECG recorder.


Caution

If you need to use the device for a long time, **please keep the lead wire connected to the device and do not plug and unplug the lead wire repeatedly!**

3. Electrodes Placement:


- Fasten the disposable electrodes into the electrode joint of lead wire.
- Remove the protective packaging on the back of the disposable electrodes.
- Place the electrodes correctly in accordance with the electrode placement illustration, and ensure the electrodes are firmly pasted on the user's skin.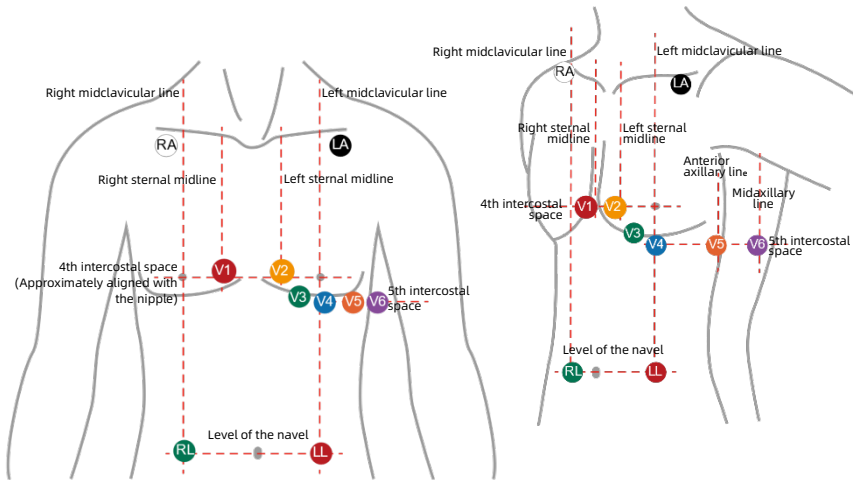

	AHA Color	AHA Label	Position
1	Red	V1	The fourth intercostal space at the right edge of sternum
2	Yellow	V2	The fourth intercostal space at the left edge of sternum
3	Green	V3	The position between V2 and V4
4	Blue	V4	Midclavicular line at the fifth intercostal space
5	Orange	V5	Anterior axillary line at the same level as V4
6	Purple	V6	Midaxillary line at the same level as V4 and V5
7	Black	LA	Left shoulder
8	Red	LL	The lower edge of rib cage, or the level of the navel at the position of middle clavicle line.
9	Green	RL	The lower edge of rib cage, or the level of the navel at the position of middle clavicle line.
10	White	RA	Right shoulder

2) Record ECG

1. Press the Enter button to wake up the screen.
2. Select "MENU" -- "Date & Time" and set the device to the current date and time.
3. Select "ESC" to return to the menu interface. Select "Options" -- "Gain" to set the appropriate recording gain.
4. Select "ESC" to return to the menu interface and select "ECG review" (Note: the device has not yet started recording ECG).
5. Press the Enter button to access the recording operation interface. Select "OK" to start ECG monitoring and recording.

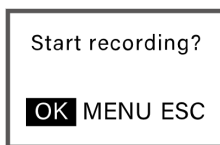6. Confirm the user ID. If the ID number needs to be modified, please modify it on this interface.
7. Please wait for the system to complete the SD card formatting automatically. After formatting, the system will automatically access the recording interface.
8. Put the Holter Recorder into a leather case and wear it with the belt or shoulder strap.
9. Recording will stop automatically after 24 hours.
10. After the recording ends, please take off the recorder according to the following steps:
 - Remove the waste battery and dispose it appropriately in accordance with the local law.
 - Press the SD card and it will automatically pop up. Then, hook the groove with your fingernail to remove the SD card.

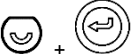
- Remove the electrodes from the user and separate the lead wires from the electrodes.

Note

- **Transfer ECG data in the SD card to the AI-ECG Analysis System,** because Holter Recorder will automatically format the SD card every time the ECG recording is started.
- **Ensure that the SD card is unlocked.**
- In the waveform interface, if no button is pressed for 15 seconds, the system will automatically switch to standby mode. Press the Enter button, the system screen will display again.


Name	Button	Function
Up button		<ul style="list-style-type: none"> ➤ Switch to the previous lead. ➤ Reduce the setting value. ➤ Move the cursor up/left.
Down button		<ul style="list-style-type: none"> ➤ Switch to the next lead. ➤ Increase the setting value. ➤ Move the cursor down/right.
Enter button		<ul style="list-style-type: none"> ➤ Enter the setting value. ➤ Enable the "Recording Operation" page. ➤ Mark the events during recording.
Down button + Enter button		<ul style="list-style-type: none"> ➤ Enable the "Exit Recording" page.

3) Install and update software


1. Choose one of the following methods to download the AI-ECG analysis system.
 - Visit the website <https://getwellue.com/pages/pc-software> to download the latest version of AI-ECG Analysis System (for Windows / Mac OS users).
 - Download the installation file from the USB drive (for Windows users only).
2. Run the installation file and select the defaults until finish installation.
3. If you need to update the AI-ECG analysis system, click "Settings" in the upper right corner of the main interface of the analysis system. When the "!" mark appears on the "Check for Updates", please manually update to the latest version.

Note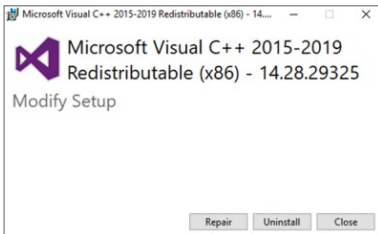

- If a similar interface pops up, please click "Close".

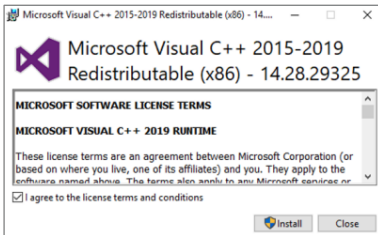- If a similar interface pops up, please click "Install".

4) Export and analyze data

1. Run the AI-ECG Analysis System.
2. Insert the SD into the card reader, and insert the card reader into the computer.

3. When the device serial number appears in the lower left corner of the software, the connection is successful. Click the "Download Data" button. Create or select a user. Then the data will automatically be imported to the analysis system and shown in the task list.

Data ID	Name	User ID	Age	Gender	E-mail	Measurement duration	Analysis State	Operation	Start Time	Download Time	Device Info
56	yuandong	2	30	Male		24 h 0 m	Analyze	View ECG	2022-05-07 15:00:06	2022-05-10 09:19:22	View
55	yuandong	2	30	Male		24 h 0 m	View report	View ECG	2018-11-21 11:02:34	2022-05-10 09:17:29	View
54	yuandong	2	30	Male		24 h 0 m	View report	View ECG	2022-05-07 18:23:00	2022-05-10 08:51:49	View
53	yuandong	2	30	Male		24 h 0 m	View report	View ECG	2022-05-07 18:15:41	2022-05-10 08:50:20	View
52	FEIZW	9	30	Male		10 m	View report	View ECG	2021-10-27 09:03:50	2022-05-09 16:37:16	View
51	zhongwen	8	30	Female		7 m	View report	View ECG	2021-12-07 13:49:28	2022-05-09 16:27:10	View
50	XIBANYA	7	30	Male		1 h 29 m	View report	View ECG	2018-10-12 23:45:32	2022-05-09 16:22:53	View
49	XIBANYA	7	30	Male		10 m	View report	View ECG	2021-12-29 17:27:05	2022-05-09 16:13:07	View
48	FAGUO	6	30	Female		54 m	View report	View ECG	2018-10-12 08:13:33	2022-05-09 15:55:59	View
47	YIDALI	5	30	Female		5 m	View report	View ECG	2021-11-04 16:56:37	2022-05-09 15:49:08	View
46	DEGUO	4	30	Male		17 m	View report	View ECG	2021-11-04 17:04:23	2022-05-09 15:41:09	View
45	EN	3	30	Female		10 m	View report	View ECG	2021-10-20 16:40:03	2022-05-09 15:32:09	View
36	yuandong	2	30	Male		21 h 41 m	Analyze	View ECG	2021-11-10 09:47:59	2022-05-09 13:57:55	View
35	yuandong	2	30	Male		12 m	View report	View ECG	2021-12-29 17:48:19	2022-05-09 13:56:47	View
34	yuandong	2	30	Male		52 m	View report	View ECG	2021-10-23 15:44:59	2022-05-09 13:38:43	View

TH12 318C

4. Click "View ECG" to view and download the complete ECG waveform.

Name: EN Gender: Female Age: 30 Measurement duration: 10 m Start measurement time: 2021-10-20 16:40:03

View Report Print Save as PDF

- (0) PVC(Premature Ventricular Contr...
- (0) Couplet of PVC
- (0) PVC Bigeminy
- (0) PVC Trigeminy
- (0) Ventricular Tachycardia
- (0) PAC(Premature Supraventricular ...
- (0) Couplet of PAC
- (0) PAC Trigeminy
- (0) Supraventricular Tachycardia
- (0) Atrial flutter
- (0) Atrial fibrillation
- ▶ (1) Overall maximum heart rate
- ▶ (1) Overall minimum heart rate

1s Lead II 2021-10-20 16:40:03 ~ 2021-10-20 16:41:03

10mm/mV Page 1 of 11

5. Click "Analyze" and register. This recording will be uploaded to the server and an ECG report will be generated.

Note

If the login interface pops up, please click "Go to login" and complete the registration.

If you need to register, please:

- Get the verification code from your email within **15** minutes and complete the registration.
 - After 15 minutes, you need to get the verification code again.
6. After the analysis is complete, click "View Report" to view and download the ECG report.

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Wellue | AI-ECG Analysis System

The screenshot shows the 'Wellue | AI-ECG Analysis System' interface. At the top, there are buttons for 'Nachrichten', 'Daten herunterladen', and 'Einstellungen'. Below these are search filters for 'Benutzer' (All Benutzer, yuandong) and 'Messdauer'. A table lists ECG records with columns: Daten-ID, Name, Benutzer ID, Alter, Geschlecht, E-Mail, Messdauer, Analysenstatus, Operation, Startzeit, Downloadzeit, and Geräteinformation. The record with ID TH12.219C is circled in red.

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