ViHealth Mobile App User Guide

English | Deutsche | Italiano | Español | Français

Contents

| | |
|------------------------------|----------------|
| User Manual..... | English 1-11 |
| Benutzerhandbuch..... | Deutsch 12-22 |
| Manuale d'uso..... | Italiano 23-33 |
| Manual de usuario..... | Español 34-44 |
| Manuel de l'utilisateur..... | Français 45-55 |

User Manual

Table of Contents

| | |
|-----------------------------------|----|
| 1. INTRODUCTION..... | 2 |
| 1.1 INDICATIONS FOR USE..... | 2 |
| 2. INSTALLATION..... | 2 |
| 2.1 DOWNLOAD THE APP..... | 2 |
| 2.2 INSTALL THE APP..... | 2 |
| 2.3 COMPATIBILITY..... | 2 |
| 3. USING THE APP..... | 3 |
| 3.1 SETTING UP..... | 3 |
| 3.2 CONNECTING TO THE DEVICE..... | 3 |
| 3.3 TRANSFER DATA..... | 4 |
| 3.4 REVIEW DATA..... | 5 |
| 3.5 REAL-TIME MONITORING..... | 9 |
| 3.6 SETTINGS..... | 9 |
| 3.7 SOFTWARE UPDATES..... | 10 |
| 4. TROUBLESHOOTING..... | 10 |

1. Introduction

1.1 Indications for Use

The ViHealth app is a mobile application for users to configure a prescribed compatible Pulse Oximeter and transfer, analyze, and display monitored data information.

ViHealth also allows the monitoring of blood oxygen saturation and pulse rate in real-time.

2. Installation

2.1 Download the App

App name: **ViHealth**

iOS: App Store

Android: Google Play

2.2 Install the App

Install the app on an Apple product or Android-powered device, including smart phones and tablets. See Compatibility for more information.

2.3 Compatibility

The ViHealth app is compatible with iOS versions 9.0+ and Android versions 5.0+.

The compatible smart device models are listed below:

| Brand | Model |
|---------|---|
| Apple | iPhone5/S, iPhone SE, iPhone6/S/Plus, iPhone7/Plus, iPhone8/Plus, iPhone X, iPhone XS, iPhone XS Max, iPhone XR, iPhone11, iPhone11 Pro, iPhone11 Pro Max |
| | iPad 5/6/7, iPad Mini 1/2/3/4/5, iPad Air 1/2/3, iPad Pro 1/2/3/4 |
| Samsung | Galaxy S5/6/7/8/9/10, Note 3/5,J7 |
| Huawei | P9/10/20/30/40, Mate 10/Pro, Mate 20/Pro, Mate 30/Pro |
| OnePlus | OnePlus 5/6/7 |
| LG | G7 |
| Google | Pixel 1/2/3/4 |

3. Using the App


3.1 Setting Up

1. Find the last four digits from the product serial number.

Note: The serial number is on the back of the device.

How to find the serial number

- 1) Detach the main unit from the wristband.
- 2) Flip the main unit over. The series number is printed on the product label.


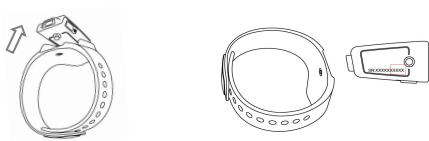
2. Take off the Pulse Oximeter after monitoring, and the data will be ready to sync once the countdown on the screen completes.
3. Make sure that Bluetooth is enabled on your smart device and run ViHealth.
4. Turn on the Pulse Oximeter and make sure the screen light is on during the connection.

3.2 Connecting to the device

1. Click the device icon when ViHealth finds your Pulse Oximeter.
2. Input the last four digits of the serial number to begin pairing.
3. Once paired, you can log onto ViHealth.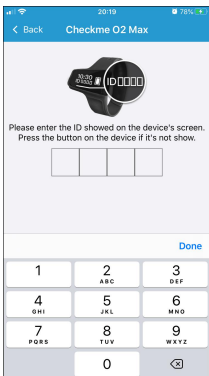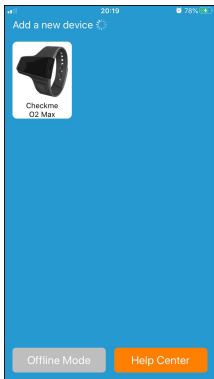

Caution: Do NOT pair the device in your smart device settings.

3.2.1 Syncing Date and Time

The date and time on the Pulse Oximeter will automatically sync when it connects to ViHealth every time.

Note: Make sure the date and time settings on your smart device is correct.

3.2.2 Offline Mode

You can log on in Offline Mode without the Pulse Oximeter, but only your history can be reviewed.


3.2.3 Help Center

For support, please click the **Help Center** on the search screen to visit our website.

3.3 Transfer Data

The Pulse Oximeter will automatically transfer the latest data to ViHealth every time it connects.


Note: Please wait for the data transmission to finish before using the App.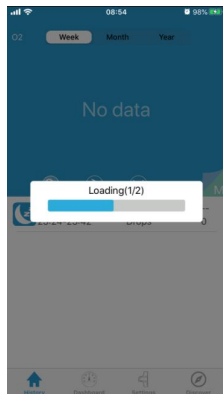

3.4 Review Data

The History screen allows you to access measurement records and trend views. You can select a recording to review more detailed information.

The items on the History screen are described below:

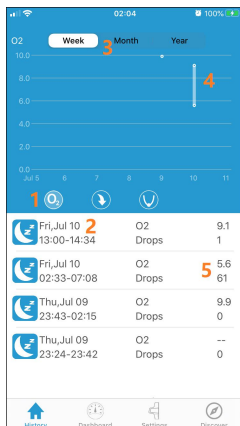1. Graphic Trend Types

There are several graphic trend types to display the trends for different parameter values.

O2 Score: The O2 score for analyzed recording data.

SpO2 Drops: The number of times the SpO2 value declines 4% from the baseline while recording.

Lowest SpO2: The lowest SpO2 value during while recording.

2. Recording Date, Time, and Duration

Displays the date, start time and end time of the current recording .

3. Time Period

There are three kinds of query time periods on Graphic Trends:

- **Week:** Displays the values for each day for the current week.
- **Month:** Displays the values for each day for the last 30 days.
- **Year:** Displays the values for each month for the last 12 months.

The trend values for the selected Graphic Trend type.

4. Trend Values

The trend values for the selected Graphic Trend type.

5. Statistics

Display the statistics for the current recording.

3.4.1 Review Recording Details

Select a recording on the Recording List, and you can see the statistics and graphic trends for the parameters.

The items on the detail recording screen are described below:

1. Statistics

Display all statistics for the current recording.

- **Recording Time:** Duration.
- **<90% Time:** Duration for when SpO2 values drop below 90%.
- **Drops over 4%:** The number of 4% drops from the baseline in the SpO2 value during the recording .
- **O2 Score:** The O2 score for the analyzed recording data. Only recordings over 1 hour will be scored.
- **Drops per Hour:** The number of times the SpO2 value declines by 4% from the baseline per hour during the recording.
- **Lowest SpO2:** The lowest SpO2 value during the recording.
- **Average PR:** The average Pulse Rate value during the recording.

2. SpO2 Graphic Trend

Displays the trend curve for SpO2 during the recording.

3. PR Graphic Trend

Displays the trend curve for Pulse Rate during the recording.

4. Motion Graphic Trend

Displays the histogram of finger motions during the recording.

5. Notes

Input a remark for this recording.

6. Help

View the online FAQ.

7. Share

Share the screenshot as a CSV file, binary file, or picture.

8. Vibration Marker

Indicates vibration events during the recording.

9. Cursor Switch

Show/Hide the cursor on the graphic trend region. You can move the cursor to view the measurement values on the trend curves.


3.4.2 Review Measured Values

To review the measured values from a certain moment during the recording:

1. Click the icon  on the bottom-right corner of the screen to show the cursor on the trend view area.
2. Move the cursor, and the measured value the cursor hovers over will be displayed.

3.4.3 Zoom Graphic Trends

To view detailed measurements during the recording. You can use pinch or spread your fingers to zoom in and out from the graphic trends on the trend view area.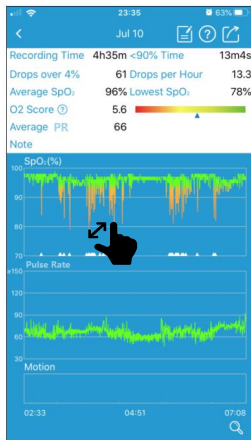

3.4.4 Share Recordings

To share the current recording and information to your doctor or friends:

1. Click the icon  on upper-right corner of the screen.
2. Choose the file format for the exported file.
3. Send the exported file.

There are three options to export the recording and information:



- **Share as CSV:** Export the data as a CSV file.
- **Share as picture:** Capture the detail recording screen as a picture file.
- **Share as binary:** Export the measurement data as a binary file.

Note: The binary file is only available to be exported on the android device.

3.4.5 Delete Recordings

To delete a recording:



1. Press and hold a recording, and a pop-up menu will display.
2. Choose “Delete” to delete the current recording.
3. If you want to delete more than one recording, choose “Select” to enter multi-select mode. Then you can choose the desired records and delete them.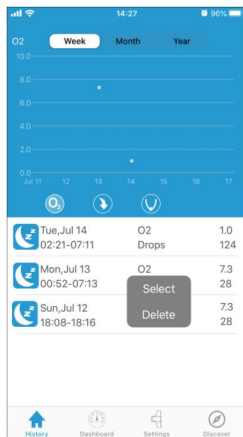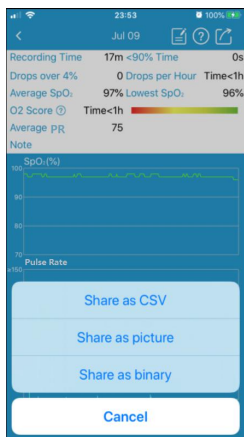

3.5 Real-time Monitoring

The ViHealth app supports monitoring SpO₂ and HR in real-time on the Dashboard screen.

Battery information for the Pulse Oximeter is displayed as well.

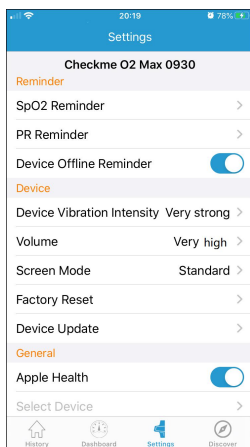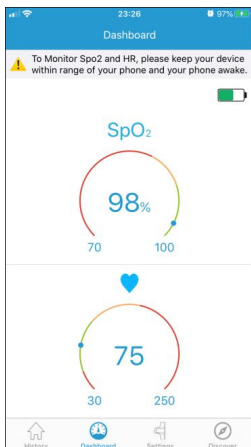3.6 Settings

The Settings screen allows you to customize the configuration for your Pulse Oximeter.

The settings menu items are described below:

- **Reminder**
 - **SpO₂ Reminder** Turn on/off the vibration or sound reminder on the device and setup the SpO₂ threshold which triggers a reminder.
 - **PR Reminder** Turn on/off the vibration or sound reminder on the device and setup the PR thresholds which triggers a reminder.
 - **Device Offline Reminder:** Turn on/off the device offline reminder on the App.
- **Device**
 - **Device Vibration Intensity:** Adjust vibration Intensity.
 - **Volume:** Setup the sound reminder volume.
 - **Screen Mode:** Setup the screen mode on the device.

Option: Standard/Always On.

For standard mode, the screen will turn off automatically when there is no operation. Then pressing the power button can wake up the screen.

For Always On mode, the screen will always light on.

- **Factory Reset:** Restore the device to factory default settings.
- **Device Update:** Update the device firmware to the latest version.

● **General**

- **Apple Health:** Enable/disable syncing measurement data to Apple Health App.
Note: Apple Health is only available for iPhone.
- **Select Device:** You can select one of your devices as the device type connected with ViHealth when you log on in Offline Mode.
- **Visit getwellue.com:** For support, please visit www.getwellue.com.
- **Help Center:** View the online FAQ for our products.
- **About:** Display the general information about Pulse Oximetry and the ViHealth app. such as the software version.

3.7 Software Updates

Please download the latest version of the ViHealth app from the Apple Store or Google Play.

4. Troubleshooting

If you experience any problems, try the following suggestions. Also check your device user guide. If you are not able to fix the problem, please contact us.

| Problem | Possible Cause | Solution |
|---------------------------------|--|--|
| The app cannot find the device. | The Bluetooth on your smart device is off. | Turn on Bluetooth on the smart device. |
| | The device screen doesn't light up. | Make sure the device is turned on and wake up the device screen. |
| My data has not been | The Bluetooth on your smart device is off. | Turn on Bluetooth on the smart device. |

| | | |
|-------------------------------------|---|---|
| transmitted. | No new data on your Pulse Oximeter | Try again after using the Pulse Oximeter to take measurements. |
| My Pulse Oximeter does not vibrate. | The Pulse Oximeter does not support vibration. | Buy a Pulse Oximeter which supports vibration. |
| | The vibration switch is off. | Check the vibration switch on the ViHealth app and turn it on. |
| Connection failure | The main unit might not be properly placed within the smart device's transmission range and is too far from the smart device. | Move the main unit within 5 m (16 ft.) of the smart device and try again. |

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Shenzhen Viatom Technology Co., Ltd.