How to use your new Rehband UD Achilles Support 807*

*different colors, sizes, thicknesses, sides

EN

Function

Gives support, compression and warmth. Improves muscle co-ordination in the ankle and lower leg areas.

Application

The support should be put on around the foot and pulled upwards so calf muscle, Achilles tendon, heel and shin bone with surrounding muscles are covered by the thicker neoprene material. knees.

Important Information

- 1) Use this product with care and only for its intended purpose. Do not make any improper changes to the product.
- 2) If you have questions regarding application, please contact your local dealer.
- 3) Do not wear products with SBR/neoprene for more than 3-4 hours without interruption.
- 4) We advise you to not use the product that contains SBR/neoprene in case of tendency to heat allergy.
- 5) Keep the product away from open flame, embers and others sources of heat. Avoid temperature above 120 °C/248 °F.
- 6) Do not continue using a product that is no longer functional, or that is worn or damaged.

Washing Instructions

- 1) Wash in warm water at 40 °C (104 °F) with similar colours.
- 2) Use laundry bag or net.
- 3) Stretch product while wet and allow to air dry.
- 4) Clean the product regularly according to care instructions on the product care label.

Size selection

- 1) Do the measurements as instructed on the packaging.
- 2) Determine the size of the support (see size chart).

Disposal

Dispose the product in accordance with national regulations.

Material

Fabric:
Polyamide(recycled yarns),elastane
Laminat:
Cover and lining: polyamide
Foam: SBR/neoprene

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