How to use your new Rehband Ankle Support – 1173* (7770*), 1172* (7973*)

*different colors, sizes, thicknesses, sides

EN

Function

Gives support, compression and warmth to the ankle. Increases blood circulation, relieves pressure and improves muscle coordination in the area. Provides stabilization.

Application

- Pull the support onto the ankle.
- Tighten the Velcro fasteners at the front as required, until the desired pressure is reached (1173*, 7770*).

Important Information

- Use this product with care and only for its intended purpose. Do not make any improper changes to the product.
- If you have questions regarding application, please contact your local dealer.
- Do not wear products with SBR/neoprene for more than 3-4 hours without interruption.
- We advise you to not use the product that contains SBR/neoprene in case of tendency to heat allergy.
- Keep the product away from open flame, embers and others sources of heat. Avoid temperature above 120 °C/248 °F.
- Do not continue using a product that is no longer functional, or that is worn or damaged.

Washing Instructions

- Wash in warm water at 40 °C (104 °F) with similar colours.
- Use laundry bag or net.
- Stretch product while wet and allow to air dry.
- Clean the product regularly according to care instructions on the product care label.
- Fasten all velcro closures (1173*, 7770*).
- Products with velcro fasteners require occasional dirt/dust removal.

Size selection

- Do the measurements as instructed on the packaging.
- Determine the size of the support (see size chart).

Disposal

Dispose the product in accordance with national regulations.

Material

Foam: SBR/neoprene or neoprene (see textile label)

Cover: polyamide and/or polyester (see textile label)

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