# How to use your new Rehband UD Runner's Knee/ITBS-Tights Men/Women – 813\*, 823\*

\*different colors, sizes, thicknesses, sides

EN

## Function

Gives support, compression and warmth. Improves muscle co-ordination in the hip and leg areas.

## Application

The thicker, neoprene material should cover bilaterally outside area of thigh and upwards as well as surrounding the knees.

## Important Information

- 1) Use this product with care and only for its intended purpose. Do not make any improper changes to the product.
- 2) If you have questions regarding application, please contact your local dealer.
- 3) Do not wear products with SBR/neoprene for more than 3-4 hours without interruption.
- 4) We advise you to not use the product that contains SBR/neoprene in case of tendency to heat allergy.
- 5) Keep the product away from open flame, embers and others sources of heat. Avoid temperature above 120 °C/248 °F.
- 6) Do not continue using a product that is no longer functional, or that is worn or damaged.

## Washing Instructions

- 1) Wash in warm water at 40 °C (104 °F) with similar colours.
- 2) Use laundry bag or net.
- 3) Stretch product while wet and allow to air dry.
- 4) Clean the product regularly according to care instructions on the product care label.

## Size selection

- 1) Do the measurements as instructed on the packaging.
- 2) Determine the size of the support (see size chart).

## Disposal

Dispose the product in accordance with national regulations.

## Material

Fabric:  
Polyamide (recycled yarns), elastane.  
Laminat:  
Cover and lining: polyamide  
Foam: SBR/neoprene

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