How to use your new Rehband Protective Support – 405*, 402*, 7752*, 7763*, 7952*, 7721*, 7727*

*different colors, sizes, thicknesses, sides

EN

Function

Gives support, compression and warmth to the knee/elbow. Increases blood circulation, relieves pressure and improves muscle coordination in the area. Provides stabilisation and impact protection.

Application

- 1) Place the support on the elbow/knee.
 - 2) The curve of the outside seams should be at the same level with the joint. Position protective pad on the tip of the elbow/knee.
- Important Information**
- 1) Use this product with care and only for its intended purpose. Do not make any improper changes to the product.
 - 2) If you have questions regarding application, please contact your local dealer.
 - 3) Do not wear products with SBR/neoprene for more than 3-4 hours without interruption.
 - 4) We advise you to not use the product that contains SBR/neoprene in case of tendency to heat allergy.
 - 5) Keep the product away from open flame, embers and others sources of heat. Avoid temperature above 120 °C/248 °F.
 - 6) Do not continue using a product that is no longer functional, or that is worn or damaged.

Washing Instructions

- 1) Wash in warm water at 40 °C (104 °F) with similar colours.
- 2) Use laundry bag or net.
- 3) Stretch product while wet and allow to air dry.
- 4) Clean the product regularly according to care instructions on the product care label.

Size selection

- 1) Do the measurements as instructed on the packaging.
- 2) Determine the size of the support (see size chart).

Disposal

Dispose the product in accordance with national regulations.

Material

Cover: polyamide or polyamide, polyester, elastane (see textile label)
Lining: polyamide or polyamide, elastane (see textile label)
Foam: SBR/neoprene or neoprene or polyurethane (see textile label)
Padding: ethylene-vinyl acetate foam (EVA foam) or SBR/neoprene and/or neoprene (see textile label)
Splints 405506: Steel

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