How to use your new Rehband Shoulder Support – 1192* (7726*), 1296* (7731*)

*different colors, sizes, thicknesses, sides

(EN)

Function

Gives support, compression and warmth to the shoulder. Increases blood circulation, relieves pressure and improves muscle coordination in the area. Provides stabilization.

Application

1192* (7726*) Place the support on your arm and shoulder joint (see picture). Pull the adjustable strap and fasten with the velcro closures on the chest at the front. 1296* (7731*) Follow the instructions on the pictures.

- 1) Place the support on your shoulder.
- 2) Retract your shoulders and close the chest strap.
- 3) Pull and attach the V-shaped closure.
- 4) Attach the arm anti-rotation strap on the adjustable arm closure. Adjust the straps if needed.

Important Information

- 1) Use this product with care and only for its intended purpose. Do not make any improper changes to the product.
- 2) If you have questions regarding application, please contact your local dealer.
- 3) Do not wear products with SBR/neoprene for more than 3-4 hours without interruption.
- 4) We advise you to not use the product that contains SBR/neoprene in case of tendency to heat allergy.
- 5) Keep the product away from open flame, embers and others sources of heat. Avoid temperature above 120 °C/248 °F.
- 6) Do not continue using a product that is no longer functional, or that is worn or damaged.

Washing Instructions

- 1) Wash in warm water at 40 °C (104 °F) with similar colours.
- 2) Use laundry bag or net.
- 3) Stretch product while wet and allow to air dry.
- 4) Clean the product regularly according to care instructions on the product care label.
- 5) Fasten all velcro closures.
- 6) Products with velcro fasteners require occasional dirt/dust removal.

Size selection

- 1) Do the measurements as instructed on the packaging.
- 2) Determine the size of the support (see size chart).

Disposal

Dispose the product in accordance with national regulations.

Material

Foam: SBR/neoprene (see textile label)
Cover: polyamide (see textile label)

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