How to use your new Rehband Support – 1222* (7722*), 1228* (7923*)

*different colors, sizes, thicknesses, sides

(EN)

Function

Gives support, compression and warmth to the elbow. Increases blood circulation, relieves pressure and improves muscle coordination in the area. Provides stabilisation and load distribution with pressure pads.

Application

- 1) 1222* (7722*) Pull the support up over the forearm and position the upper part of the support above the elbow joint. The pads should be positioned about 5 cm below the elbow joint (below the affected muscle insert). Close and tighten the Velcro strap until you feel the desired pressure from the support (it should still be possible to put a finger under the strap).
- 2) 1228* (7923*) Place the support on your forearm about 5 cm below the elbow joint. Position the pad of the support below the affected muscle insert. Tighten the strap until you feel the desired pressure from the support (it should still be possible to put a finger under the strap).

Important Information

- 1) Use this product with care and only for its intended purpose. Do not make any improper changes to the product.
- 2) If you have questions regarding application, please contact your local dealer.
- 3) Do not wear products with SBR/neoprene for more than 3-4 hours without interruption.
- 4) We advise you to not use the product that contains SBR/neoprene in case of tendency to heat allergy.
- 5) Keep the product away from open flame, embers and others sources of heat. Avoid temperature above 120 °C/248 °F.
- 6) Do not continue using a product that is no longer functional, or that is worn or damaged.

Washing Instructions

- 1) Wash in warm water at 40 °C (104 °F) with similar colours.
- 2) Use laundry bag or net.
- 3) Stretch product while wet and allow to air dry.
- 4) Clean the product regularly according to care instructions on the product care label.
- 5) Fasten all velcro closures.
- 6) Products with velcro fasteners require occasional dirt/dust removal.

Size selection

- 1) Do the measurements as instructed on the packaging.
- 2) Determine the size of the support (see size chart).

Disposal

Dispose the product in accordance with national regulations.

Material

- 1222* (7722*) Foam: SBR/neoprene or neoprene (see textile label)
1222* (7722*) Cover: polyamide and polyamide-elastane (see textile label)
1228* (7923*) Polyamide, polyurethane, thermoplastic polyurethane
Padding: neoprene

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Rehband Limited