How to use your new Rehband Support – 1252* (7759*), 1253* (7754*), 1258* (7757*)

*different colors, sizes, thicknesses, sides

EN

Function

Supports and provides compression and warmth to the knee joint. Increases blood circulation, relieves pressure and improves muscle coordination in the area. Stabilizes the joint area. 1258*: Reduces the pressure selectively.

Application

- 1) Pull the brace up to the knee.
- 2) The patella cut-out should be centered on the patella.
- 3) 1252*: the pad inside should be placed on the inner or outer side of the knee joint. Place the straps cross-wise (as shown in the picture). Tighten the strap until you feel the desired pressure.
- 4) 1258*: Place the patella stabilizer right below the kneecap (as shown in the picture). Tighten the strap.

Important Information

- 1) Use this product with care and only for its intended purpose. Do not make any improper changes to the product.
- 2) If you have questions regarding application, please contact your local dealer.
- 3) Do not wear products with SBR/neoprene for more than 3-4 hours without interruption.
- 4) We advise you to not use the product that contains SBR/neoprene in case of tendency to heat allergy.
- 5) Keep the product away from open flame, embers and other sources of heat. Avoid temperature above 120 °C/248 °F.
- 6) Do not continue using a product that is no longer functional, or that is worn or damaged.

Washing Instructions

- 1) Wash in warm water at 40 °C (104 °F) with similar colours.
- 2) Use laundry bag or net.
- 3) Stretch product while wet and allow to air dry.
- 4) Clean the product regularly according to care instructions on the product care label.
- 5) Fasten all velcro closures 1252* (7759*), 1258* (7757*).
- 6) Products with velcro fasteners require occasional dirt/dust removal.

Size selection

- 1) Do the measurements as instructed on the packaging.
- 2) Determine the size of the support (see size chart).

Disposal

Dispose the product in accordance with national regulations.

Material

Foam: 1252* (7759*), 1253* (7754*) SBR/neoprene (see textile label)
Cover: 1252* (7759*), 1253* (7754*) polyamide; 1258* (7757*)
polyamide and polyamide-polyurethane (see textile label)

Padding: 1252* (7759*) ethylene-vinyl acetate (EVA); 1258* (7757*)
silicone sponge rubber (see textile label)

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