# How to use your new Rehband Support – 135\* (7790\*), 132\* (7791\*)

\*different colors, sizes, thicknesses, sides

EN

## Function

Gives support, compression and warmth to the knee/elbow. Increases blood circulation, relieves pressure and improves muscle coordination in the area. Provides stabilization. Suitable for strength training and lifting heavy weights. This product comes with semi-flexible straps for personal adjustment and strong positioning.

## Application

- 1) Place the support on the knee/elbow.
- 2) The Rehband logo on the support should be positioned at the same level as the middle of the joint.
- 3) Tighten the straps until you feel the desired pressure from the support.

## Important Information

- 1) Use this product with care and only for its intended purpose. Do not make any improper changes to the product.
- 2) If you have questions regarding application, please contact your local dealer.
- 3) Do not wear products with SBR/neoprene for more than 3-4 hours without interruption.
- 4) We advise you to not use the product that contains SBR/neoprene in case of tendency to heat allergy.
- 5) Keep the product away from open flame, embers and others sources of heat. Avoid temperature above 120 °C/248 °F.
- 6) Do not continue using a product that is no longer functional, or that is worn or damaged.

## Washing Instructions

- 1) Wash in warm water at 40 °C (104 °F) with similar colours.
- 2) Use laundry bag or net.
- 3) Stretch product while wet and allow to air dry.
- 4) Clean the product regularly according to care instructions on the product care label.
- 5) Fasten all velcro closures.
- 6) Products with velcro fasteners require occasional dirt/dust removal.

## Size selection

- 1) Do the measurements as instructed on the packaging.
- 2) Determine the size of the support (see size chart).

## Disposable

Dispose the product in accordance with national regulations.

## Material

Foam: SBR/neoprene (see textile label)  
Cover: polyamide (see textile label)

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