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
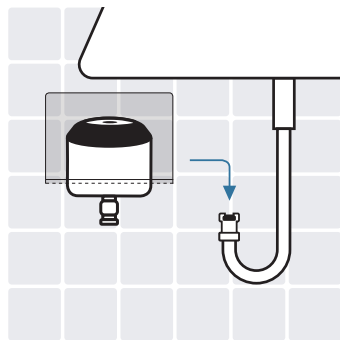
2 FILTERINSTALLATION

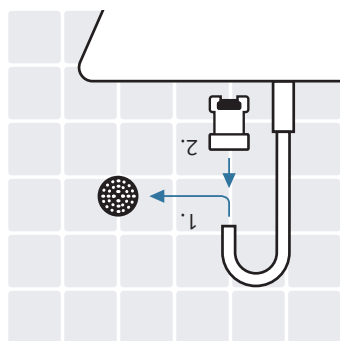1. Remove the aerator. 2. Install quick-release coupling with seal.

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Quick-release coupling M24, external thread (incl. seal), with AquaStop, (Art. No. 100298)

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Quick-release coupling M22, internal thread (incl. seal), with AquaStop, (Art. Nr. 100296)

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Further adapters and functional materials on request

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49,3 mm

52 mm

The Safety Tap NEO is a sterile, point-of-use tap filter for use in high-risk areas of medical facilities for the prevention of bacterial nosocomial infections caused by *Pseudomonas aeruginosa*, *Legionella pneumophila*, *Stenotrophomonas maltophilia* or *Staphylococcus aureus* in immunocompromised and immunosuppressed patients. The effect is achieved by interrupting the transmission path of these germs through filtration of the water and protection against retrograde contamination when using the water taps through hygienic design and antimicrobial protection functions.

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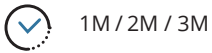
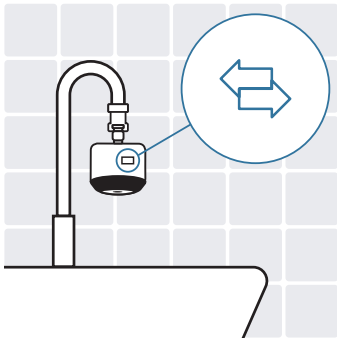
Note next change date on change label. Filter life see filter label/packaging.

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After the service life has expired (1M, 2M or 3M), uninstall the filter.

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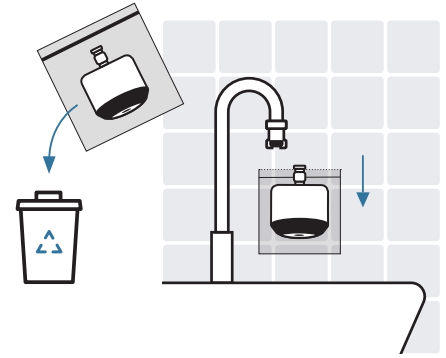
Put the pressure seal bag over the filter and uninstall the filter. Dispose of filter in general waste.

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The Safety Tap NEO requires no maintenance within the specified operating time. The outside of the filter can be cleaned using common surface disinfectants. To prevent retrograde contamination, daily wipe disinfection of the Safety Tap NEO is recommended.

When replacing the Safety Tap NEO, a small amount of unfiltered water will flow out. Over time, this will lead to deposits. For this reason, adapters and connections should be checked at regular intervals for functionality and proper condition and replaced as required.

In the case of a deviation from the specifications or a damage of the package, the product should be returned to Sanquell, along with the "Defect Report" form (download at sanquell.de).

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Flow rate 15.5 l/min at 5 bar, 13.1 l/min at 3 bar

Max. operating pressure 5 bar⁴

Max. operating temp. 60 °C (at 75 °C ≤ 90 min over operating time)

Connection Quick-release coupling

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2 Measured in accordance with the international ASTM F838-15a standard – Standard Test Method for Determining Bacterial Retention of Membrane Filters Utilized for Liquid Filtration. 3 Depending on water quality. If you use the Aqua free filter change service, the filter will be changed every 4 or 8 weeks. 4 The Safety Tap NEO must not be connected to low pressure units (boiler).