

# SAFE EYES

I wear MARKUS T


## SAFE EYES

I wear MARKUS T Protective Eye Shield / Protective Face Shield

## SAFE EYES

Certification EN 166:2001, CE marking

- + Panoramic protective shield for free view in all directions
- + Protection against splashes, coarse dust, wind
- + Safe and comfortable due to ergonomic design
- + Temple for individual adjustment
- + Quick and easy protective shield change
- + Comfortable to wear over mouth-nose masks

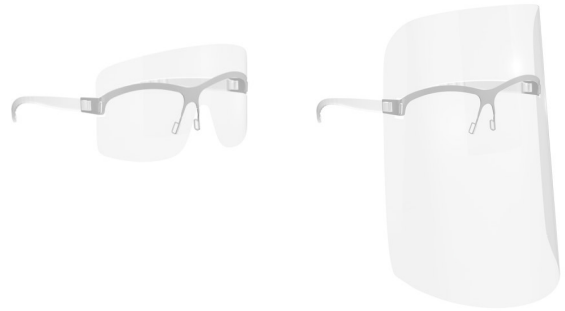

SAFE EYES were developed for personal protection during the COVID-19 threat. They meet the basic health and safety requirements of the PPE regulation (EU) 2016/425.

SAFE EYES are designed to protect against the transmission of the corona virus and reduce its spread. The application is in the medical, nursing, rescue and other fields of work affected by COVID-19, which require the same protective purpose. SAFE EYES are designed to protect persons involved in the containment of the spread of the corona virus against hazards. Therefore, SAFE EYES are suitable for areas of the public life and the world of work.

### Range of application and properties

Application areas	Reduction of ocular, oral and nasal hazards: splash protection for medical work, laboratory work, work with liquids, cooking, cleaning work, painting, gardening, outdoor sports
-------------------	--

Protection level	PSA Category II
------------------	-----------------

Frame	Stainless steel
-------	-----------------

Protective Shield	Polycarbonate, Transparent Optical class 1: fatigue-free, permanent use
-------------------	--

Temple	Polyamide Anatomical adaptation: Heat at high temperature, then shape.
--------	---

SAFE EYES GmbH  
An der Manufaktur 1  
33334 Gütersloh  
GERMANY

safe-eyes.de  
contact@safe-eyes.de

Man. Director : Markus Temming  
AG Gütersloh HRB 9248  
USt.-ID-Nr. DE 285964384  
St.-Nr. 351/5775/1850

# SAFE EYES

I wear MARKUS T


Eyewear Cord Polyester, Black  
infinitely variable

---

Comfort Extremely light, suitable for wearing over small prescription glasses or contact lenses,  
quick putting on and taking off, best ventilation

---

Assembly Remove and insert foil

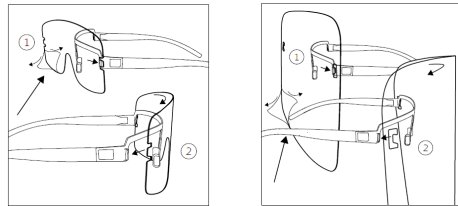

## Cleaning, care and storage

Cleaning Clear water, household detergent, avoid strong mechanical cleaning

---

Disinfection Spray and wipe the frame and protective shield with commercially available hand  
disinfectant or heat in the oven at 80°C for 20 minutes.  
Reuse after disinfection is at the discretion of the user.

---

Care Change protective shield as soon as scratches, damages or changes are visible.

---

Storage Case or shipping box  
The period of use is 1 year after date of manufacture.

---

# SAFE EYES

I wear MARKUS T

## Labelling

Not every protective shield is marked. The marking, the complete manufacturer's address and the date of manufacture can be found on the packaging unit.

Frame

MAT 1 CE

## Notes

Protective shields alone do not necessarily provide sufficient protection, but they provide good additional protection and increase the safety of the user. Further precautions and additional protective equipment such as face masks or Safety Glasses are essential depending on the area of work.

Before using your SAFE EYES, make sure that they are intact and suitable for your application needs.

Only use original accessories and spare parts with your SAFE EYES.

SAFE EYES can only guarantee the intended safety if it is properly installed and properly adjusted. Removable parts must not be carried separately.

SAFE EYES is designed as eye and face protection against splashes and does not meet the requirements of safety glasses against mechanical effects, impacts, splinters, mechanical parts, acids, gases, burning materials, sparks, irradiation, flames, radiation.

Exclusion of liability:

When used for the described purpose, SAFE EYES meet the requirements of Regulation (EU) 2016/425

for personal protective equipment and the requirements of the European harmonised standard EN 166 : 2001 (partially applied). This eye protection product has been tested and certified by the notified body: ECS GmbH, Hüttfeldstraße 50, 73430 Aalen, Germany (NB 1883) For applications in the medical environment the product fulfils the requirements the regulation for medical devices (EU) 17/745 class 1.

The materials used for the face shield are not known to be harmful to the user or cause skin irritation. However, materials that come into contact with the wearer's skin may cause allergic reactions in susceptible individuals.

Manufactured and placed on the European market:

Markus Temming GmbH | An der Manufaktur 1 | 33334 Gütersloh | Germany  
Phone: +49 (0) 5241.74343-0 | Fax: +49 (0) 5241.74343-10  
[info@markus-t.com](mailto:info@markus-t.com) | [www.markus-t.com](http://www.markus-t.com)

SAFE EYES GmbH  
An der Manufaktur 1  
33334 Gütersloh  
GERMANY

[safe-eyes.de](http://safe-eyes.de)  
[contact@safe-eyes.de](mailto:contact@safe-eyes.de)

Man. Director : Markus Temming  
AG Gütersloh HRB 9248  
USt.-ID-Nr. DE 285964384  
St.-Nr. 351/5775/1850